# CO2 laser beauty

Gynecology

Fractional

Cutting

3 in1 multifunctional  
CO2 laser beauty equipment  
power reaches 40w  
professional medical level


### Stylish appearance and excellent use feeling

□□□□□□ □□ □□□□□□□□ □□□□□□ □□□□□□ □□□□□□ □□ □□□□□□.

## South Korea imported light guide arm

7 joins articulated light guiding arm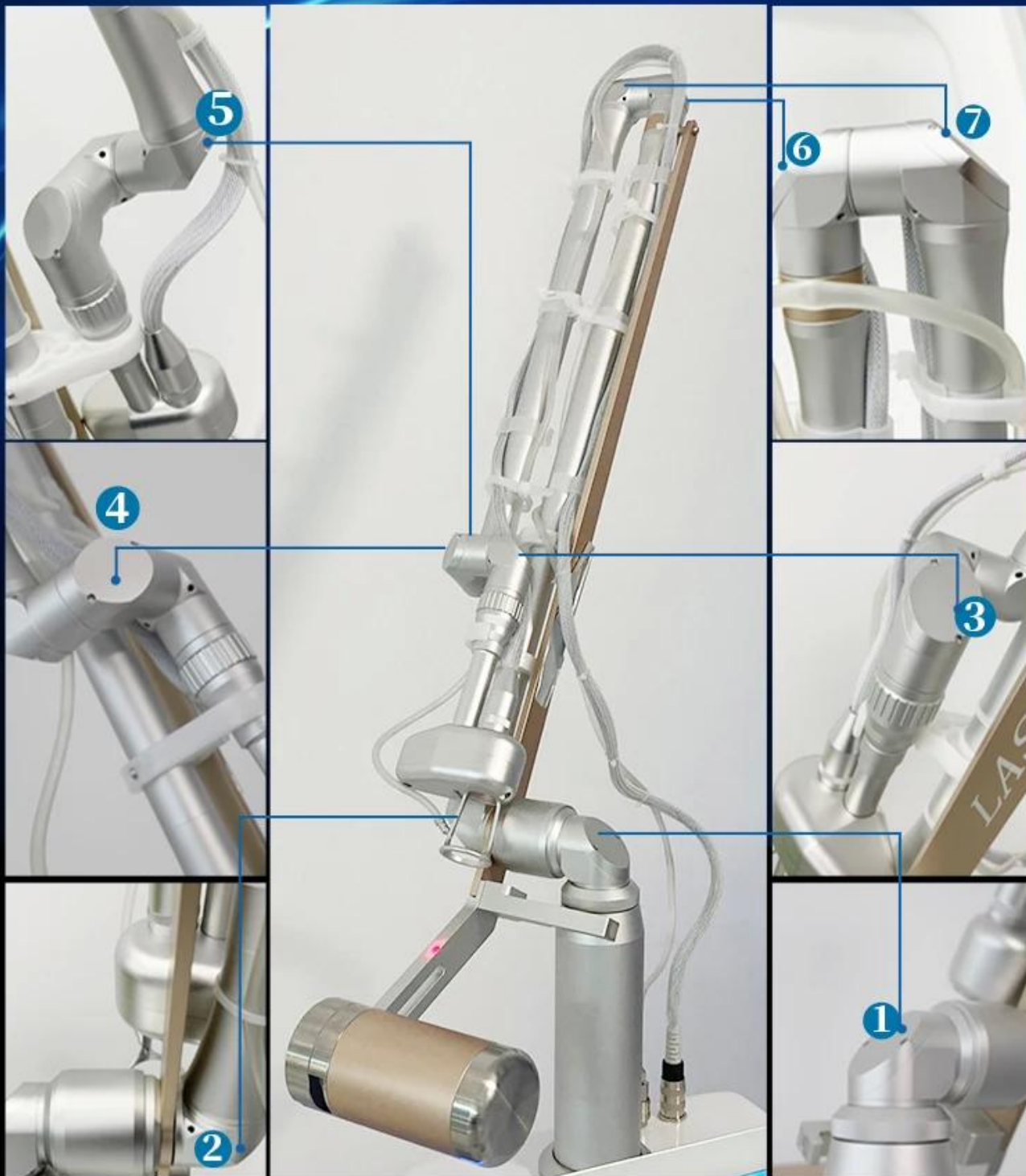

000000 00000000 000 00000 00 0 00000 000 00000 00 00000 00 00000.  
 000000000 00000 00 00000 000000 00000000 00000000


[illegible]

Page 10 of 10


### • Trachea

When working, gas is sprayed out, which has the functions of cooling, cooling and dust removal.

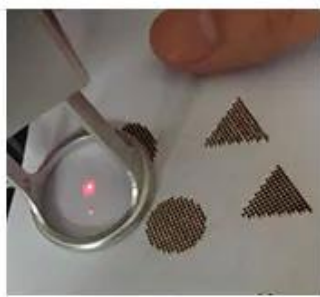### Adjustable dot shape and size
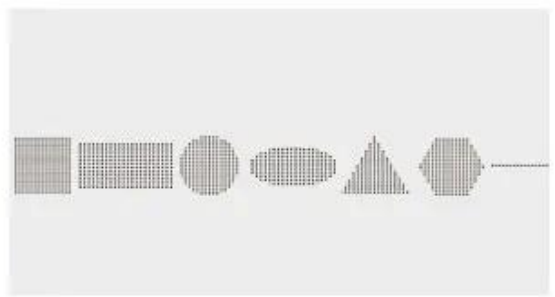
### Shape display

### • Rf tube

Good stability and high safety


### • Metal frame

In addition to the shell, the all-metal internal structure is firmer and has long service life.

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# WORKING PRINCIPLE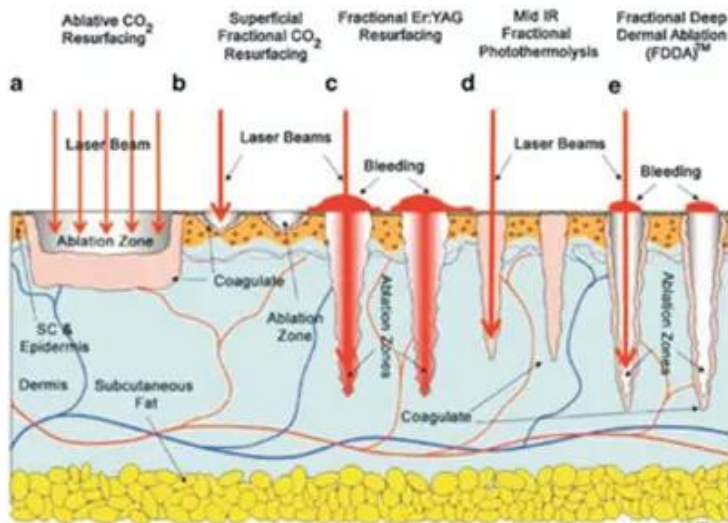

The CO<sub>2</sub> laser uses a unique head piece which tracks the CO<sub>2</sub> 10.6  $\mu$ m wavelength light as it passes light through its optical lens to penetrate the skin. We can control the depth of the penetration from only a few micrometers in depth (only as deep as a few sheets of paper) to much deeper with tiny thermal channels. This will also determine the length of healing, the number of treatments and the cost. Each thermal channel creates a small micro-injury but does not significantly disturb or disrupt the surrounding tissue.

These microscopic injuries caused by the laser (around 15-20% of the treated area) are the beginning of the healing process. By creating thousands of these perforations underneath the skin, your epidermis layer starts to heal from the edge of these microscopic perforations very quickly. By healing so fast it remodels, tightens and stimulates the collagen, your skin tightens, which in turn smooths out lines, and significantly improves your skin's tone and complexion.

□□□□□□ □□□□□□

# TREATMENT EFFECT T

before treatment

during treatment

after treatment

after 30days

Acne Scars

Burn Scars

Hypertrophic Scars

Surgery Scars

Chicken Pox Scars

Accident Scars

□□ □□□

## Simple and practical operation interface

Fractional cutting private multi-function multiple modes of operation  
4 scan modes, sequence, random, continue and ultra pulse

□ □ □ □ □   □ □ □   □ □ □ □ □

○○○○ ○○○○ ○○○ OEM ○ ○○○○○○ ○○○○ ○○○○○○ ○ ○○○○○○○○ ○○○ ○○○ / fre / ○○○○○○

[illegible]