

EXFU·FIND S



COMMIT TO BE FIT
GET SCULPTED WITH
EMSCULPT

LESS Fat
Muscle MORE

EXCLUSIVELY AVAILABLE AT IPAL CLINIC



EXFU·FIND S



EMSCULPT

25%
more muscle,

30%
less fat,

now all the way to your flanks



*Uses high intensity electromagnetic technology, which has the ability to induce supramaximal muscle contractions.

*During treatment, the muscles are contracted to supramaximal levels. Intensive supramaximal

contractions enhance release of epinephrine which triggers a cascade effect leading to supramaximal lipolysis in fat cells.

*Meanwhile, responds with a deep remodeling of its inner structure, induces the growth of myofibrils (muscle hypertrophy), the creation of new protein strands and the growth of muscle fibers (muscle hyperplasia). The process results in increased muscle density and volume.

ADVANTAGE

► Patented cooling system

- It ensures the continuous operation of the machine, Continuous and stable energy output at up to 7 Tesla.



WHO NEEDS



Bodybuilder



Weight loss person



Model&actor




Postpartum mother

OPERATING PARTS

The location of the handle operation must be consistent with the interface selection location



Work principle

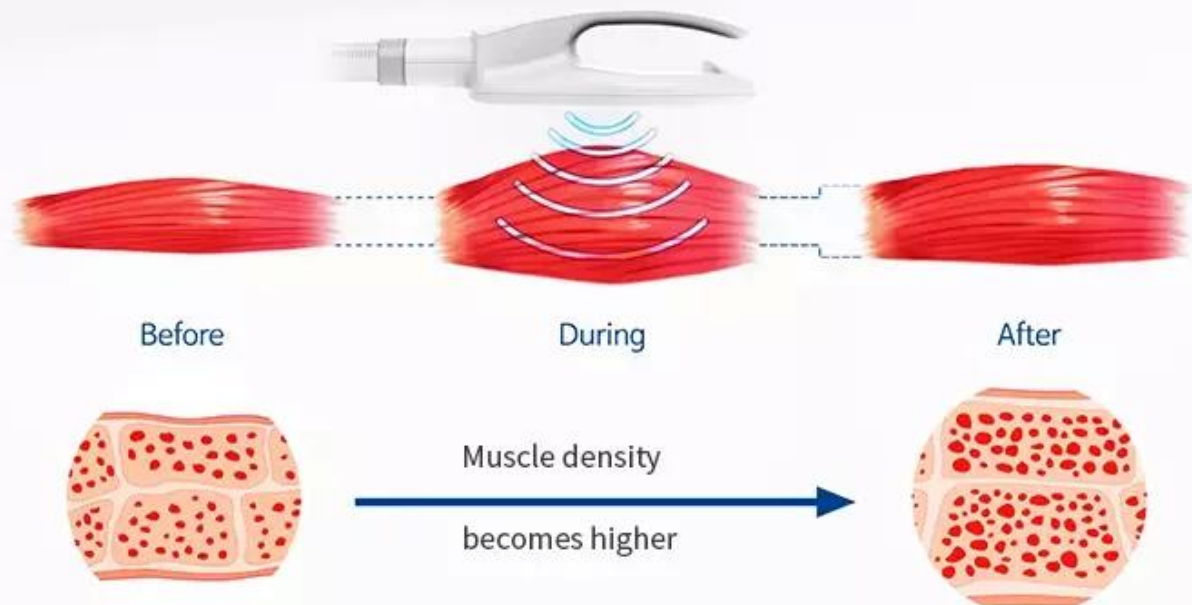

Man

Build Muscle	Course	1	2	3	4	5
	Program	Hypertrophy	Combo2	Hypertrophy	Combo2	Combo2
Fat Burner	Course	1	2	3	4	5
	Program	HIIT	Combo1	Strength	Strength	Combo1

Build Muscle	Course	1	2	3	4	5	6
	Program	Hypertrophy	HIIT	Combo2	Hypertrophy	HIIT	Combo2
Fat Burner	Course	1	2	3	4	5	6
	Program	HIIT	Combo2	Strength	HIIT	Combo2	HIIT



Women



1. ALL frequency procedures of magnetic thin are thin are designed according to the feeling and effect of actual exercise.
2. The basic 30-minute plan in cludes:a 1-minute "stretch mode",a 5-minute "warm-up mode", four 5-minute "workout mode" and a 4-minute "cool mode" set.
3. Each group is basically a step-by-step configuration, ideally for weight training, designed to provide e best results in frequency and inteusity for each individual's different purposes.

EMS MUSCLE STIMULATOR

-19%

Estimated fat
reduction

+16%

Estimated increase in
muscle mass

30 MIN

Save time and
effort

30,000 SPORTS

compared to 30,000
crunches/squats



AMAZING EFFECT

Before & After



Before

Five weeks later

Eight weeks later



Before

After



Before

After



Before

After



Before

After

Specification

Model Number	FIND S
Warranty	2 years
After-sales Service Provided	Other, Field maintenance and repair service, Online support, No after-sales service, Free spare parts, Video technical support, Field installation, commissioning and training
Type	STANDING
Feature	Other
Plugs Type	Other
Application	For Home Use
Target Area	Body
Product name	Weight loss Increase muscle machine
Chassis material	3D blister film
Magnetic strength	14 Tesla
Maximum output power	3000w
Input power	100-240V/50HZ
Screen	15.6-inch high-definition touch screen
Certification	CE, ISO, MEDICAL CE

Contact us! Learn more!